Bedienungsanleitung
Instruction manual
Manual de instrucciones
Mode d'emploi

Econ XXL


DE

EN

ES

FR

Enjoy mobility.

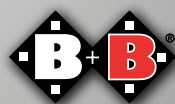Inhalt

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Notizen | Notes | Notas | Notes

Contents

1.	Preliminary remark	46	5.	Asseccories	66
1.1.	Signs and Symbols	47	5.1.	Anti-tipping wheels	66
2.	Important Safety Instructions	47	5.2.	Adjustable back	66
2.1.	General Safety Instructions	47	5.3.	Height-adjustable side panel	66
2.2.	Anti-tip wheels (optional)	49	5.4.	Legrest horizontally adjustable	66
2.3.	Safety in vehicles	50	5.5.	Amputation leg support	67
2.4.	Use on Public Roads	50	5.6.	Infusion/oxygen bottle holder	67
2.5.	Brakes	50	5.7.	Cane holder	67
2.6.	Purpose	51	5.8.	Drum brake	67
2.7.	Indications for use	51	5.9.	Safety belt	67
2.8.	Contraindications	52	6.	Use	68
2.9.	Declaration of conformity	52	6.1.	Getting in and out from the side	68
2.10.	Responsibility	52	6.2.	Getting in and out from the front	68
2.11.	Service life	52	6.3.	Lap belt	69
3.	Product and delivery overview	53	6.4.	Climbing stairs and high steps	70
3.1.	Delivery check	53	6.5.	Driving	71
3.2.	Content of delivery and check of content	53	7.	Technical data	73
3.3.	Typeplate and serial number	54	8.	Care instruction for the user	76
3.4.	Product overview	55	8.1.	Cleaning	76
4.	Assembly / Adjustments	56	8.2.	Disinfection	77
4.1.	Folding and unfolding	56	9.	Check before use	78
4.2.	Leg rests	57	9.2.1.	Brake check	78
4.2.1.	Horizontally adjustable leg rests (optional)	59	9.2.2.	Inspection of tires	78
4.3.	Amputation legrest (optional)	59	9.2.3.	Maintenance note	78
4.4.	Side parts	60	9.2.4.	Maintenance advice for the specialist trade	79
4.4.1.	Height adjustable side parts (optional)	60	10.	Repairs	79
4.5.	Brakes	61	11.	Storage/dispatch	79
4.5.1.	Drum brakes	62	12.	Passing on the wheelchair	80
4.6.	Seat height	63	13.	Disposal	80
4.7.	Seat angle	64	14.	Transport	80
4.8.	Back height	64	15.	Warranty conditions	81
4.9.	Wheels and tyres	65			
4.10.	Wheelbase extension	65			

1. Preliminary remark

Dear User,

With the purchase of your Bischoff & Bischoff XXL-wheelchair Econ XXL you have received quality product which sets standards through its various adjustment features.

These operating instructions intend to help you to familiarise yourself better with the operating and the handling of your wheelchair.

Before using your new wheelchair for the first time, please read and follow the instructions in this manual carefully. The instruction manual is an integral and necessary part of the wheelchair. This is why you should keep it close by and include it when passing on the wheelchair to someone else. For visually impaired users, this document is available as a PDF file at our website www.bischoff-bischoff.com.

Any repairs and adjustments require special technical training and may therefore only be carried out by authorized Bischoff & Bischoff dealers.

1.1. Signs and Symbols

Warning! These are especially safety-relevant instructions. Follow the directions in the manual!

Important! This is especially useful information on a given subject.

2. Important Safety Instructions

2.1. General Safety Instructions

- To avoid falls and dangerous situations, you should first practice driving your new wheelchair on level, unobstructed ground. We recommend having an assistant with you.
- Please note there is a risk of entrapment between moving parts when adjusting, using and maintaining the wheelchair.
- The accompanying person should be physically and mentally capable of operating a wheelchair.
- Never use the wheelchair under the influence of alcohol or other drugs.
- Please make sure that all parts have been properly installed before using the wheelchair.
- Never lift the wheelchair by parts that are not fixed firmly (the armrests or legrests).
- Make sure that the screwing axle at the wheels are tightly in the axle holder, otherwise there is risk of injuries.
- Please note that displacing the center of gravity by shifting the body, or by adding loads may increase the risk of tipping over.
- Please have an assisting person for help when you have to get over obstacles or steps. Use the tip assist (see overview of the wheelchair) to tip the wheelchair appropriately.
- Steps and larger obstacles may only be crossed with the help of at least two assisting persons. For this, use lifting points 1 and 2 (see overview of the wheelchair chap. 7.4). If equipment such as ramps, lifts or step climbing aids are available, these should be used (pic. A).
- When driving on an incline/slope, it is necessary to be aware that a risk of tipping exists even at low angles, depending on the axle position setting. Use of tip protection is recommended (pic. C).
- Avoid using on uneven, loose surfaces.
- When getting into or out of the wheelchair, apply both parking brakes.
- All road traffic regulations must be observed when driving on public roads.

- The passive lighting (reflectors) of your wheelchair must always be visible to other road users.
- The seat and upholstery materials meet the requirements for resistance to flammability in accordance with EN 1021- 2 / ISO 8191-2, but do not expose upholstery or other parts to sources of fire - such as cigarettes.
- The wheelchair must only be used to transport one person. The wheelchair must not be used to transport goods or more than one person.
- You must not use the handbrake to stop the wheelchair.
- When braking at fast speeds or on long downward slopes when using the push rims, fingers and hand surfaces become hot. Caution! Danger of burning (pic. C)!
- We recommend that leather gloves are used for outdoor use. These increase grip and protect fingers and hands from dirt and injury at the same time.
- Please note that the seat and backrest upholstery will heat up if directly exposed to sunlight and may cause injuries to your skin when touched. You should therefore cover these parts or protect the wheelchair from exposure to direct sunlight.
- Also note that the frame and upholstery parts may cool down significantly in winter. If possible, do not park the wheelchair outdoors in cold weather.
- The dimensions of the wheelchair exceed the recommended values of DIN EN 12183:2014/A.1 and PRM TSI Annex M in terms of size and manoeuvring range.

Please note: Due to the dimensions of the wheelchair, it is necessary to check whether your wheelchair can also be moved in the planned environment. This applies in particular to the use of escape routes.

- both parking brakes are locked.
- the side parts for getting in/out are swung backwards.
- the foot plates (for getting in/out) are folded upwards, swing inwards/outwards or are removed.

Reporting of incidents:

- If you, as the operator, user or their relatives, discover possible defects
- or functional limitations, please contact an authorised specialist dealer
- immediately. Operators, users or their relatives should inform their specialist dealer who provided the product of any suspected serious incidents that have had or may have had, directly or indirectly, any of the following consequences and which affect them. You can also report
- this to the competent federal authority.
- the death of a patient, user or other person,
- the temporary or permanent serious deterioration of the state of health of a patient, user or other person,
- a serious threat to public health.

2.2. Anti-tip wheels (optional)

The anti-tip wheels prevent the wheelchair from tipping backwards. Please note, driving with anti-tip wheels on uneven ground, lift platforms and ramps is only possible to a limited extent. Ensure sufficient clearance above, below and on the sides.

When transferring between wheelchair and bed, always ensure:

2.3. Safety in vehicles

You can tell whether or not your wheelchair is approved as a seat in a motor vehicle for transporting mobility-impaired persons by the symbol on the type plate on the wheelchair:

- Wheelchair suitable as a seat in a motor vehicle for transporting persons with reduced mobility (Fig. D)
- Wheelchair not suitable as a seat in a motor vehicle for transporting persons with reduced mobility (Fig. D)

The Econ XXL wheelchair is not suitable as a vehicle seat, therefore it must not be used as such under any circumstances. It does not meet the requirements of ISO 7176-19. Failure to do so may result in serious injury and death in the event of an accident.

Do not use the wheelchair as a vehicle seat it may result in serious injury and death in the event of an accident.

2.4. Use on Public Roads

The wheelchair is designed for indoor and outdoor use. Please keep in mind that you are using public roads and therefore have to observe all road traffic regulations.

Do not endanger other road users through inconsiderate driving, especially on walkways.

2.5. Brakes

Service brake:

When driving, brake the wheelchair using the handrims. Please note that your hands can get hot when doing this.

Parking brake/parking:

As soon as the wheelchair is stationary, push both brake levers forward. (see chapter 4.5). The wheelchair is safely braked when it can no longer be pushed away and both brake levers are pushed forward as far as they will go. Pull the lever back towards you to release the brake (see chap. 4.5).

Drum brake:

The wheelchair can optionally be equipped with a drum brake that can only be operated by an attendant via the brake levers on the push handles.

2.6. Purpose

The wheelchair is designed solely to increase mobility and to transport adult people with impaired mobility in accordance with indications listed. Unless otherwise indicated, the instructions in this operating manual are intended for the occupant as the operator.

The maximum user load is 250 kg.

Please note:

We are only able to issue a guarantee if the product is used in accordance with the specified conditions and for the specified purpose.

2.7. Indications for use

Walking disability or distinct walking impairment due to:

- paralysis
- loss of limb
- extremity defect/deformation
- joint stiffness/damage (not affecting both arms)
- other diseases.

2.8. Contraindications

It is improper to use the wheelchair for adults, if there is:

- a perception disorder/ perceptual disturbance
- a disturbance of equilibrium
- extremity loss on both arms
- joint stiffness/damage on both arms
- sitting incompetence
- reduced or insufficient eyesight

2.9. Declaration of conformity

Bischoff & Bischoff GmbH, as the manufacturer, declares under its sole responsibility that the Econ XXL wheelchair complies with all requirements of MDR 2017/745 that are applicable.

2.10. Responsibility

We shall only provide a warranty if

- the product is used under the specified conditions and for the intended purpose,
- modifications, extensions, repairs and maintenance work are only carried out by persons who have been authorised by us to do so, and
- the wheelchair is used in compliance with all instructions for use.

2.11. Service life

If used as intended, and the safety, care and maintenance instructions are observed, the expected service life is up to 5 years. Beyond this time, the wheelchair can be used continuously, provided the product is in a safe condition.

3. Product and delivery overview

3.1. Delivery check

All Bischoff & Bischoff wheelchairs are subjected to a proper final inspection in our house and are provided with the CE mark.

The Bischoff & Bischoff wheelchairs are supplied folded in a special carton. After unpacking, please keep the cardboard if it is possible. You may need it later for storage or for the return of the wheelchair.

Supply, assembly and instruction of use of the wheelchair are done by a qualified medical devices dealer.

If delivered by rail or a forwarding agent, the product shall be checked at once, with regard to transportation damage at once in the presence of the shipper.

Check the content for completeness and intactness. If there are irregularities and damages do not hesitate to call our customer service.


3.2. Content of delivery and check of content

After receipt of the goods, please check the content for completeness right away. The content consists of:

- outer package
- instruction manual
- pre-assembled wheelchai

3.3. Typeplate and serial number

The Type Plate (fig. 01) and the serial number are located on the bottom side of the wheelchair.

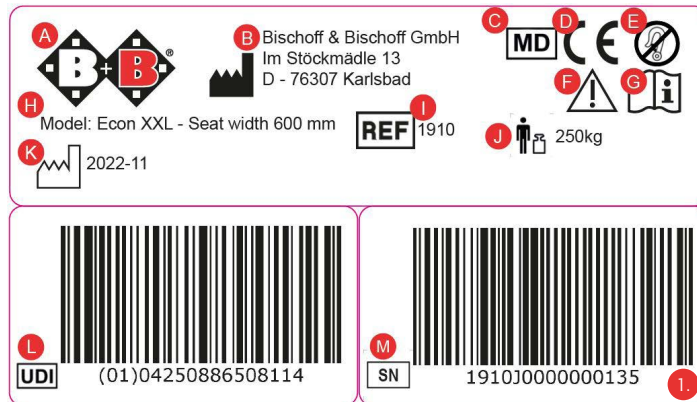- A** Manufacturer logo
- B** Manufacturer information
- C** Medical product
- D** CE mark
- E** Follow instructions regarding securing the product in the vehicle
- F** Caution! - Follow instructions for use
- G** Important - Adhere to the operating instructions
- H** Model designation
- I** Model number
- J** Maximum weight capacity
- K** Date of manufacture
- L** EAN 13 item number
- M** Serial number

3.4. Product overview

Product overview Econ XXL (fig. 2):

4. Assembly / Adjustments

You can carry out the adjustments of your wheelchair yourself with a little manual dexterity. If you have any queries, please contact your mobility dealer.

Please be careful of your fingers when making adjustments. Risk of entrapment!

4.1. Folding and unfolding

Delivery

Your new Bischoff & Bischoff wheelchair is delivered assembled and folded in the original carton by Bischoff & Bischoff.

To avoid damages during transportation, parts which have to be stuck on are delivered separately (fig. 03).

Unfolding

Put your wheelchair next to you. Tip it as far as necessary to relieve the drive wheel. Now push with your flat hand the seat pipe which is closer to you and unfold the wheelchair (fig. 04).

Then swing the push bar onto the handles and fix it with the hand screw (fig. 5).

Please be careful of your fingers when unfolding. Danger of pinching.

Folding

Place your wheelchair next to you. Fold the foot plate upward and undo the Velcro fastener of the back cover under the seat. Loosen the hand screw on the right handle and swing the push bar down (fig. 06).

Now pull the seat cover upward until the chair is completely folded, and press the armrests together (fig. 07).

Please be careful of your fingers when unfolding. Danger of pinching.


4.2. Leg rests

The Econ XXL is equipped with removable leg rests (fig. 08).

Both leg supports are supplied with the foot plates attached. The foot plates on both leg supports can be folded upwards in the sitting position. In addition, a continuous footrest is available for you to use.

Continuous footrest

You can easily fit the continuous footrest onto both foot plates (fig. 9). You can easily pull the footrest off again to get in and out.

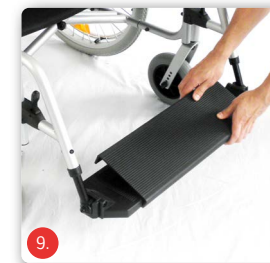Assembling the leg rests

Hold the legrests on the upper part and bring it into the leg rest fixtures so that they point outwards. Swing the leg rests forward until they lock in place (fig. 10).

Before you start driving your wheelchair make sure that the leg rests are firmly fixed.

Swinging the leg rests

To make it easy for you to get on and off the wheelchair you can fold up the foot plates or swing the leg supports outside or inside without removing them.

- Pull the button, in this way, the lock is opened (fig. 11).
- Now you can swing the leg rest outwardly or inwardly.
- If necessary, fold the foot plate upward beforehand.

To get back to the original state just swing the leg rests frontally until they snap in.

When you get off your wheelchair make sure you NEVER step on the foot plate under no circumstances!

Adjustment of the leg rest length

The length of the leg rest can be adjusted up to 12 cm. To do this you have to undo the socket-head cap screw (hexagon socket 5 mm) and adjust the leg rest according to your choice (fig. 12).

The maximum extension length is marked!

4.2.1. Horizontally adjustable leg rests (optional)


These are attached, detached, swung away and adjusted as the standard leg rests are (fig. 13).

Horizontal adjustment functions:

- Loosen lever screw.
- put the leg rest into the desired position by raising or lowering the foot plate bracket tube
- Re-tighten lever screw (fig. 14). Make sure the legrest is firmly seated.

Make sure that both legrests are set at the same height and that the adjustment screws are tightened!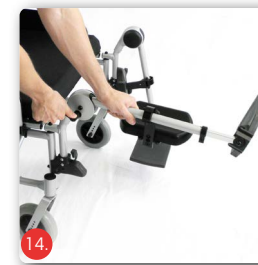

4.3. Amputation legrest (optional)

Fitting, removing, swivelling and adjustment are carried out as with the standard legrests (fig. 15).

The cushion for the amputation legrest can be adjusted:

- Loosen the fixing screw (fig. 16) underneath the cushion.
- Move the cushion into the required position.
- Re-tighten the screw; make sure it is firmly seated.

4.4. Side parts

Swinging the side parts

To make it easier to get on and off the wheelchair, both side parts with the armrests can be swung backwards. Use the lock device lever (fig. 17), press it gently with your finger and at the same time swing the side part backwardly (fig. 18).

Locking the side parts

To lock the side part, fold it forward and let it snap into the lock device with a gentle push (fig. 19).

Remove side part

Loosen the rear fastening of the side part by turning the ring through 180° (fig. 20). The grip is now facing inwards. Now push the locking lever at the front in order to be able to remove the side part by pulling upwards.

4.4.1. Height adjustable side parts (optional)

It is possible to equip your Econ XXL wheelchair with height adjustable side parts. The height-adjustable side parts can be fitted, removed and swivelled in the same way as the standard side parts.

Height adjustment:

- Press the pushbutton (fig. 21) in order to move the armrest into the required position.
- When you have reached the required position, release the pushbutton. The locking device then clicks into place.
- The height has then been correctly selected, if the upper arm and forearm are more or less at right angles when the forearm is laid on it.

4.5. Brakes

Using the parking brake

To use the brake just push the brake lever forward until it snaps (fig. 22). To loosen the brake, pull the lever backwards (fig. 23).

Adjustment of the parking brake

If you should notice any unusual braking behaviour you have to do the following:

- First check the tyres for adequate air pressure. There should be a pressure of 300- 400 k Pa (3-4 bars) (does not apply to tyres which are puncture proof tyres).
- Check the tread depth. Pay attention whether there is even abrasion. If the tread depth is > than 1 mm, the tyres must be replaced by a qualified mobility dealer.
- Now check the adjustments of the brake. It must be adjusted in the way that there is a distance between the running surface of the wheel and the pivot bolt of 12 mm when brakes are open. The pivot bolt must be horizontal e.g. parallel to the ground.
- After loosening the Allen screw with a hexagon socket (5 mm) on the brake mount (fig. 24), you can position the brake correctly.
- The parking brakes effect only one driving wheel at a time.
- Pay attention to have the same air pressure on both driving wheels to achieve a directional stability.

- Once the adjustment process is completed, firmly tighten the loosened screw up again and carry out a brake test!
- The brake system should be adjusted only by a qualified mobility dealer.

In general !

Since the drum brake is the most important safety device on your wheelchair we recommend the use of a drum brake when an assisting person is frequently pushing the wheelchair, the drum brake is optionally available. Besides the drum brake allows deceleration.

At a drum brake the braking shoe does not effect the tire, but brakes in the wheel hub, this braking system is largely independent of the tyre condition (air pressure).

4.5.1. Drum brakes

Using the drum brakes by an accompanying person

The drum brake is operated by pulling both brake levers simultaneously (Fig. 25). Remember that both sides must be pulled at the same time to prevent the wheelchair from skewing sideways.

If you want to park the wheelchair, operate the brakes until the wheelchair comes to a standstill. Then lock the knee lever parking brake (Fig. 22) to prevent the wheelchair from rolling away.

Adjustment of the drum brakes

The basic adjustment of the drum brake is carried out on the rope clamping screw. The final adjustments are carried out with an adjusting nut on the brake cable. If the adjusting nut is turned up, the braking efficiency is increased.

- The drum brakes affect only one of the drive wheels at a time!
- Make sure that the drive wheels have the same air pressure to achieve directional stability.

- Drum brakes get worn out as well. If the braking performance is reduced, the wheelchair must be withdrawn from circulation. To avoid this because of safety reasons your braking device should be checked by a qualified mobility dealer every six months!
- After the adjustment are made, please tighten all the screws and perform a brake test!

In general!

The adjustment of the brake system should be done only by a qualified mobility dealer!

4.6. Seat height

There are 5 positions for tailoring the seat height to the user's requirements. Driving wheel and steering wheel must be in the same position in each case.

Adjustments of the seat height can affect the stability of your wheelchair; a lower seat height has a positive impact on stability.

Driving wheel

- First remove the rear wheels to facilitate the adjustments.
- Loosen the screws on the adapter (Allen key, SW 4 and ring spanner SW 10), (fig. 26), and pull them out completely.
- Reinsert the screws into the adapter and tighten them up again.
- The locking brakes must then be readjusted!

Steering wheel

- Loosen the axle (Allen key, SW 4), (fig. 27), and take it out completely.
- Place the steering wheel in the right position for the required seat height.
- Reinsert the axle into the steering fork and firmly tighten it up again.

4.7. Seat angle

When changing the seat height, the seat angle must not be changed. The height adjustment on the drive wheel and on the steering wheel must always be made at the same distance. The seat angle must always remain the same.

4.8. Back height

The back height can be adjusted to the user's requirements in 5 positions. Remove the screws on the lower part of the backrest tube and adjust the backrest height to the desired level (Fig. 28).

Seat height adjustments can affect the stability of your wheelchair, a lower seat height has a positive effect on stability.

4.9. Wheels and tyres

The spoked wheels are tightened with a power socket wrench (M15) on the chassis suspension (fig. 29).

Drive wheels

Your „Econ XXL“ has puncture-proof PU tyres with tubes size 24 x 1,75“ as standard equipment. The tyres must be replaced by your authorised dealer when they are worn or faulty.

Steering Wheels

Your Econ XXL has puncture proof PU steering wheels without a valve 7“ in diameter (fig. 30). Faulty or worn out wheels must be changed by an qualified mobility dealer.

Before driving off, check that the tyres have sufficient tread depth.

4.10. Wheelbase extension

The wheelbase can be extended by 80 mm by turning the retaining plates. To do this, remove the two screws (Fig. 31) and turn the retaining plates backwards. Please note, the retaining plate of the right drive wheel is to be mounted on the side of the left drive wheel and vice versa the same for the retaining plate of the left drive wheel.

5. Accessories (optional)

5.1. Anti-tipping wheels

Always use the anti-tipping wheels in pairs.

To assemble the anti-tipping wheels (fig. 29) proceed as follows:

- First detach the rubber profiles of the tilt aid from the frame.
- Push the round profile of the anti-tipping wheels on the side frame pipe.
- Now bolt the tilt protection with a screw M 6 x 35 and a self-locking nut M6 in the designated drilling on the frame.

5.2. Adjustable back


On request, your wheelchair can also be equipped with an adaptable back for more comfort.

5.3. Height-adjustable side panel

If required, it is also possible to equip your wheelchair with a height-adjustable side section. You can find more information about this in chapter 4.3.1.

5.4. Legrest horizontally adjustable

If you need a horizontally adjustable legrest, this is also available as an accessory. You can find more information in chapter 4.2.1.

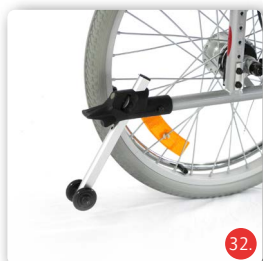5.5. Amputation leg support

An amputation leg support with adjustable pad is also available as an accessory, see chapter 4.2.2 for a description.

5.6. Infusion/oxygen bottle holder

An infusion/oxygen bottle holder is available for transport.

5.7. Cane holder

A cane holder is available as an accessory for stowing a walking stick.

5.8. Drum brake

The Econ XXL wheelchair can optionally be ordered with a drum brake. A subsequent conversion is not possible.

5.9. Lap belt

If required, a safety belt is available as an accessory. This is fitted onto the side frame of the wheelchair with the side part either temporarily removed or swung away. Take the open end of the belt once around the side frame tube and then fix it with the belt buckle.

You also use the belt buckle to adjust the length of the belt. Adjust the belt length so that the belt is tight, but does not apply pressure.

6. Use

6.1. Getting in and out from the side

- Move the wheelchair as close as possible parallel to the other seat.
- Should you want to transfer from or to another wheelchair or rehabilitation stroller, secure this one first by activating the parking brake.
- Secure the Econ XXL against rolling away accidentally by activating the parking brake.
- Pivot the armrest backwards and away, or remove the side part with the armrest.
- Fold the foot plates/footrest to the side or remove the leg rests.
- Now slide sideways onto the other seat.
- Make sure that you are sitting as far back in the seat as possible.
- Finally, return the leg rests to their original position and reinsert the side parts if necessary.

Please take care of your fingers when using it. Danger of pinching!

6.2. Getting in and out from the front

The parking brake must be applied before entering/exiting the vehicle. Trips outside the home should only be undertaken with sufficient driving experience. If possible, a person should accompany you.

- At first fold up the footrests or take off the leg rests.
- Move the Econ XXL and the current or intended seat together front to front if possible and decelerate the Econ XXL.

- Should you want to transfer from or to another wheelchair or rehabilitation stroller, secure this one first by activating the parking brake.
- Now slide sideways onto the other seat.
- Then hook the leg rests back in their initial position and pivot the foot plates downwards.

Never climb onto the foot plates! Danger of tipping!

6.3. Lap belt

Your wheelchair can be equipped with a lap belt. This is fitted to the side frame of the wheelchair with the side section temporarily removed or swung down. Wrap the open end of the belt once around the side frame tube and then fix it with the belt buckle. The belt length is also adjusted using the belt buckle. Adjust the belt length so that the belt sits tightly but does not pinch.

The lap belt must always be fastened and closed while using the wheelchair.

6.4. Climbing stairs and high steps

- Helpers must only take hold of securely fitted parts of the frame. The lifting points are shown in figures 30 and 31.
- When going up stairs, the helper behind the wheelchair pulls the wheelchair over the step, holding it by the securely mounted handles.
- The helper who is below takes hold of the front frame tubes and stabilises the position. At the same time, he pushes the rear wheels of the wheelchair against the steps.
- When going down stairs, the helper who is below brakes by pushing the wheelchair right against the steps. The helper above holds the wheelchair at the push handles, secures it and holds it in the right position. The wheelchair should roll down step by step.

The holding points for transporting the wheelchair are exclusively the frame and the push handles. Armrests must never be used to lift the wheelchair!

For obstacles and stairs with more than three steps, two attendants must help.

6.5. Driving

Position yourself in the seat of your Bischoff & Bischoff wheelchair. Ensure that you are sitting as far back in the seat as possible.

Moving and braking are achieved via the push rims on the drive wheels. Be careful when taking your first rides until you get familiar with your new wheelchair and its driving ability.

Wheelchairs have just limited tipping or slipping security. You have to be extra careful when braking, approaching or turning round on slopes or downward slopes. If you lean forward too much this will increase the danger of tipping, too. To make your wheelchair safer, all wheelchairs can be equipped with a seat belt (not standard equipment).

You can hang shopping bags or a backpack on the backrest frame.

To overcome obstacles easier, the accompanying person shall bring the wheelchair in the right angle to the obstacle, by pushing the foot on the tipping aid (fig. 36).

An accompanying person may only push the wheelchair by the push handles (Fig. 34), not by the handlebar, as there is a risk of injury.

The static stability of the basic model is 5° (ca. 8 %).

The critical obstacle height at the basic model in unfavourable situations is 23 mm (maximum centre of gravity shift).

The critical obstacle height is the height in which the centre of gravity (with the user) is right vertical over the obstacle rim. If the user tries to put the wheelchair with the help of the driving wheels on the obstacle without having shifted the centre of gravity forwardly, the wheelchair will tip over backwardly.

NE

Note that the risk of tipping backwards increases significantly when bags and rucksacks are attached.

The wheelchair exceeds the recommended width of 700 mm. Mobility restrictions, e.g. at doors, may therefore occur.

Attention: Due to the dimensions of the wheelchair, it is necessary to check whether your wheelchair can also be used in the planned environment. This applies in particular to the use of escape routes.

According to the Road Traffic Act 66/1 of 1st January 1981, hand driven wheelchairs which take part in public road traffic (even on pavements) must have lighting.

Econ XXL is equipped with reflectors (passive lighting)

- 2 red reflectors at the back
- 2 yellow reflectors at both sides

If you use the wheelchair after nightfall you need additional lights (active)

- a white light at the front
- a red light at the back

The functioning of these lights must be assured through a driving independent electrical circuit (batteries).

Before driving off, check that the lighting system is in working order.

7. Technical data

Dimensions

Total length with leg rests (mm):	1100
Width (mm):	820 - 920
Height (mm):	895 - 945
Folded (BxHxL) (mm):	375 x 980 x 890 (1100)
Lower leg length (mm):	400 - 525
Height of handle (mm):	880
Seat width (mm):	600, 650, 700
Seat depth (mm):	480
Seat height front (mm)	
without cushion:	470 - 520
with cushion:	525 - 575
Seat height rear (mm)	
without cushion:	420 - 470
with cushion:	475 - 525
Seat back angle (°):	12
Seat angle (°):	+4,6 to max. +0,5
Seat back height (mm):	460 - 535 without seat cushion
Height of the armrests (mm):	230
Length of the armrests (mm):	320
Width of the armrests (mm):	55
Front armrest attachment (mm):	430
Push rim Ø (mm):	500
Position of the rear axle with respect to the backrest (mm):	70
Knee angle (°):	70
Wheel camber (°):	0

Weigths

Max. weight capacity (kg):	250
Unloaded weight (kg):	34,7 - 38
Weight without attachments (kg):	26,1 - 28,8
Weight of attachments	
- 2 x Legrest (kg):	2,2
- 2 x Side part (kg):	2,6
- Seat cushion (kg):	3,8
- Footboard	0,6

Driving characteristics

Obstacle height (mm):	130
Turning radius (mm):	860
Ground clearance (mm):	225
Steering range (mm):	1300
Stability	
Nominal gradient (°):	7

Operating conditions

Temperature (°C):	-20 + 50
Air humidity (%):	20 - 80

Storage conditions

Temperature (°C):	-20 - +65
Air humidity (%):	45 - 60

Tyres

Drive wheels:	24 x 1,75", 61 x 4,5 mm PU
Steering wheels (mm):	180 x 45 PU

Equipment

Folding mechanism:	doubled crossbrace
Side parts::	desk, can be swung away and removed (optionally height adjustable)
Leg rests::	can be removed / swung away with continuous footrest, the lower leg length is adjustable (optionally adjustable horizontally)
Brakes:	parking brake (compact brake) drum brake for accompanying person.
Axle driving wheels:	axial drive wheel M15
Frame:	steel, powder-coated against corrosion
Components:	steel, powder-coated against corrosion
Seat and back cover:	foam with nylon cover flame resistant
Armrest:	PU foam
Acessories:	Horizontally adjustable leg supports, Amputation leg support, Height adjustable Side panels, Drum brake, Anti-tip/anti-tip castors, Adjustable back, Height-adjustable side panel, Infusion/oxygen bottle holder, Cane holder, Lap belt

The dimensions stated are below the manufacturing tolerances of +/- 10 mm.

8. Care instruction for the user

8.1. Cleaning

The seat and back materials can be cleaned with warm water using a sponge and/or a soft brush. In case of stubborn dirt, a commercial cleaning agent can be added to the water.

Note: Never use aggressive cleaning agents such as solvents / scouring agents or hard brushes.

Plastic parts are best cleaned with a commercially available plastic cleaner. Please observe the product information of the cleaning agent.

- Plastic claddings are attacked by non-ionic surfactants and solvents, especially alcohols!
- Wipe the frame parts of the wheelchair with a damp cloth. For heavier soiling, also use a mild cleaning agent.
- Clean the wheels with a damp brush with plastic bristles (do not use a wire brush!).
- Do not use a high-pressure cleaner.
- Wash the push handles, arm pads, seat and back pads with a mild detergent.
- Upholstery parts can be machine washed at 40°C. Do not machine dry, only dry at room temperature.

8.2. Disinfection

- Before disinfection, the rollator must be thoroughly cleaned. The wheelchair can be disinfected with a household disinfectant. Use approved spray disinfectants or alcohol-based wipe-on disinfectants.
- In the case of multiple product users, a commercial disinfectant is prescribed.
- Use only wipe-on disinfectants approved by the Robert Koch Institute (RKI) and the Association for Applied Hygiene (Verbund für Angewandte Hygiene e.V.) (e.g. Bacillol AF or Kohrsolin FF); see manufacturer's recommendations for exposure time and concentration.
- Caution: Damaged upholstery parts cannot be sufficiently disinfected. These should be replaced.
- Under certain circumstances, the surfaces can be affected by disinfectants. Longer-term restrictions of the functionality can be the result.

Please note our enclosed overview of the most important care, hygiene and maintenance tasks.

Attention: Danger of infection! If there are several users, the wheelchair must be cleaned after each use.

Attention! Do not use a high-pressure or steam cleaner.

Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your specialist dealer will be happy to provide you with these.

9. Check before use

9.2.1. Brake check

Make sure your brakes are operating properly before each use.

It should not be possible to push the wheelchair away while the brakes are activated.

The wheelchair should be easy to drive without grinding noises and with good direction stability when the brakes are released.

If you notice unusual behaviour when braking, inform your mobility dealer immediately and do not use the wheelchair.

9.2.2. Inspection of tires

Before each journey, check that the tyres are in good condition. If the tyre is damaged, have it repaired by an authorised specialist workshop.

9.2.3. Maintenance note

We recommend that the wheelchair is inspected by a specialist dealer at least once a year.

In the event of faults or defects on the wheelchair, it must be handed over immediately to the specialist dealer for repair.

The repair and replacement of parts is carried out by the specialist dealer. The necessary information and documents for repair and maintenance will be made available to the specialist dealer on request.

If you have any questions, please contact the Bischoff & Bischoff customer service.

You must check your wheelchair regularly according to the maintenance schedule and find it to be in perfect working order.

9.2.4. Maintenance advice for the specialist trade

Maintenance must be carried out in accordance with the Bischoff & Bischoff maintenance plan. A maintenance schedule will be provided upon request. A training programme for the specialised trade is also available.

10. Repairs

Repairs and maintenance measures may only be carried out by authorised partners (medical supply stores or specialist dealers) can be found. To find a partner near you, please call our customer service.

11. Storage/dispatch

If the wheelchair is going to be stored or transported, all removable and not tightened parts, except of the drive wheels, have to be removed and packed in separate cartons. The separately packed parts can be packed together in a larger carton.

It is recommendable to keep the original packaging for this purpose, so that it is available in case you need it. In that way your wheelchair is

protected against environmental influences during storage or transport. For the storage of your wheelchair, please note the specifications in the technical data. To avoid the formation of mould and damage to the upholstery parts, the storage location should be as dry as possible and not be exposed to direct sunlight.

12. Passing on the wheelchair

The wheelchair is suitable for reuse. Before it is passed on to another user, it must be serviced and hygienically prepared by a specialist dealer. When passing on the wheelchair, please remember to give this user manual and the address of your specialist dealer to the new user.

13. Disposal

If your wheelchair is no longer needed and you want to dispose of it, please contact your authorized dealer.

If you want to dispose of the waste yourself, ask local recycling companies about the regulations in your area.

14. Transport

You can tell that your wheelchair is not approved as a seat in motor vehicles for the transport of mobility-impaired persons (KMP) by the symbol on the type plate (Fig. 37):

The wheelchair is not crash tested and must therefore never be used as a seat in a KMP.

Your wheelchair should only be transported when folded (Fig. 38). When folded, the wheelchair should be secured to fixed frame parts in the vehicle.

15. Warranty conditions

- For products supplied by Bischoff & Bischoff, the period of warranty is 24 months from purchase date. The products are delivered free from production and material defects. In case of evidence of material or manufacturing defect, defective parts will be replaced free of charge.
- This warranty excludes claims, if a repair or replacement of a product or a part is required for the following reasons:
 - normal wear and tear, which include but is not limited to the following parts where fitted; Batteries, motor brushes, handles, armrests, upholstery, tyres, brakes, caps, etc.
 - overloading the product such as, for example, by exceeding the maximum user weight or load.
 - the product or part has not been maintained or serviced in accordance with the Instruction manual, the care and hygiene instructions or the manufacturer's recommendations as shown in the maintenance instructions.
 - Accessories have been used which are not original accessories.
 - The product or part has been damaged through negligence, accident or incorrect use.
 - changes/modifications have been made to the product or parts, which deviate from the manufacturer's specifications.
 - Repairs have been carried out, before our customer service has been informed of the circumstances.
- To enforce the warranty, please contact your dealer with the exact details of the nature of the difficulty. The product must be repaired by a dealer authorised by Bischoff & Bischoff.
- For parts, which have been repaired or exchanged within the scope of the warranty, the warranty will be extended to the remaining warranty period for the product in accordance with point 1).
- For original spare parts which have been fitted at the customer's expense, these will have a 12 months guarantee, following the fitting.

6. If the repair fails after a reasonable time, the customer can, at his discretion, request a reduction, compensation or the termination of the purchase contract.

7. The guarantee is subject to the law of the country in which the product was purchased from Bischoff & Bischoff & Bischoff. Please also observe the Bischoff & Bischoff care, warranty, hygiene and maintenance instructions. Your dealer will be happy to provide these to you.
For replacement devices and repairs, the warranty period is one year.

Non-observance of the operating instructions as well as improper use, maintenance jobs and technical changes or adaptations without the permission of the Bischoff & Bischoff company, lead to guarantee loss and the loss of product liability in general.

Warranty certificate

Product: Econ XXL

Serial number:*

* (To be filled in by the dealer)

Dealer:

Date and seal

(Pay attention to our general terms and conditions.)

*Signs are located on the seat bracket

Note!

Print errors, product and colour changes and technical information are excepted. The colours of the products shown may be slightly different in real life. The images may show accessories. The current version of each Instruction manual is ready for download on our website:

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