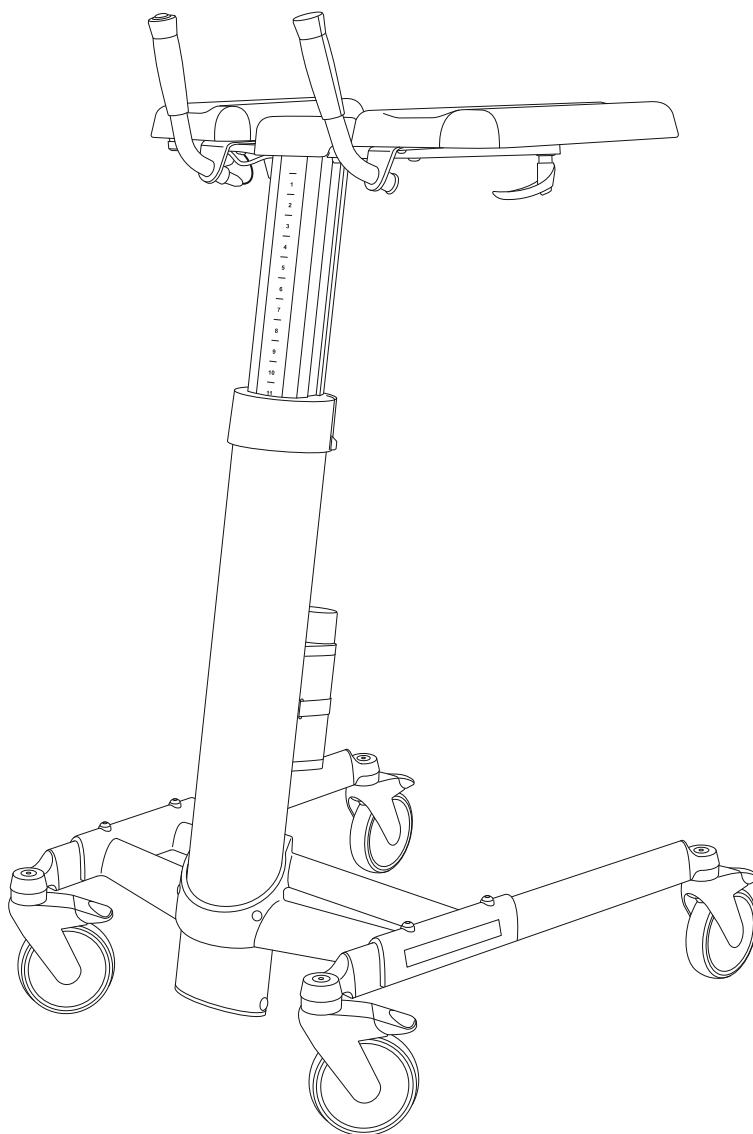
# TOPRO

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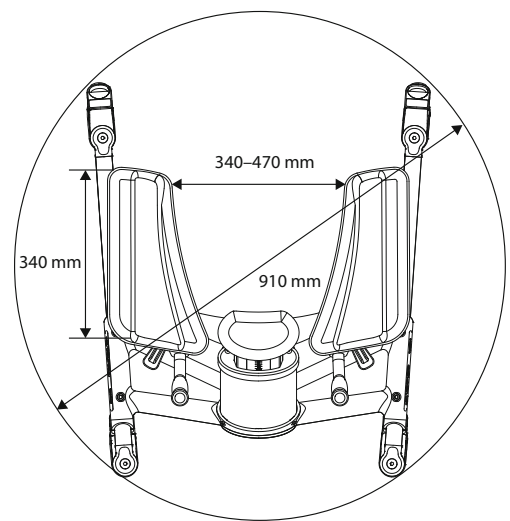
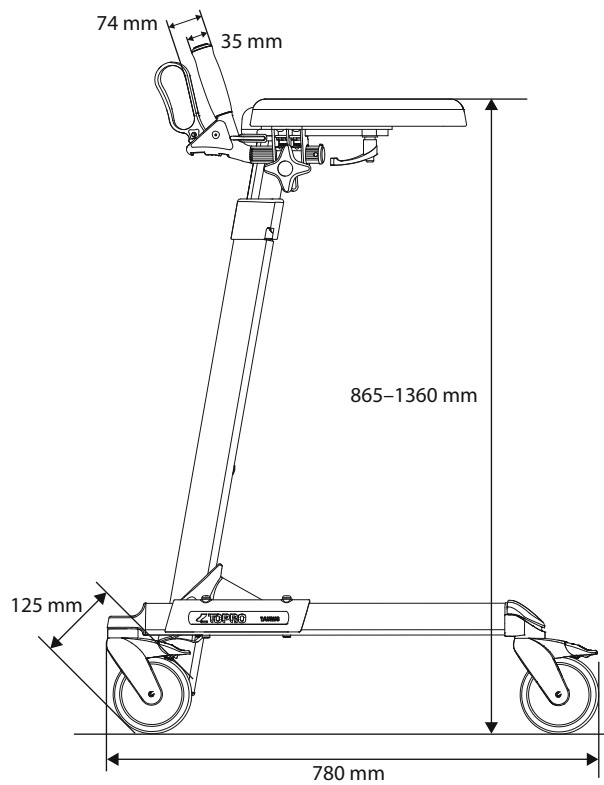
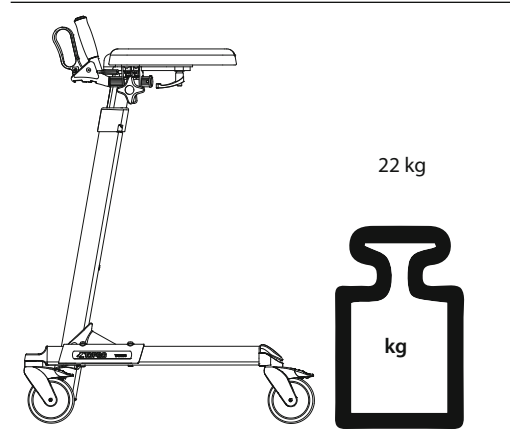
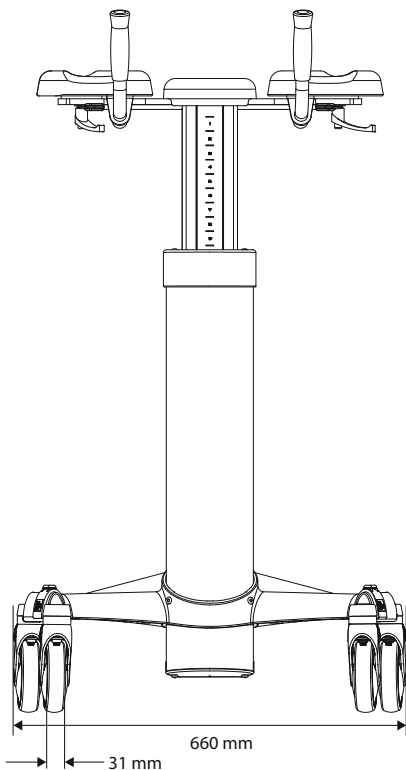
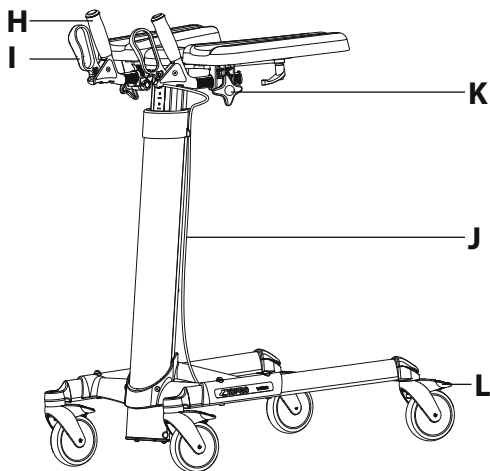
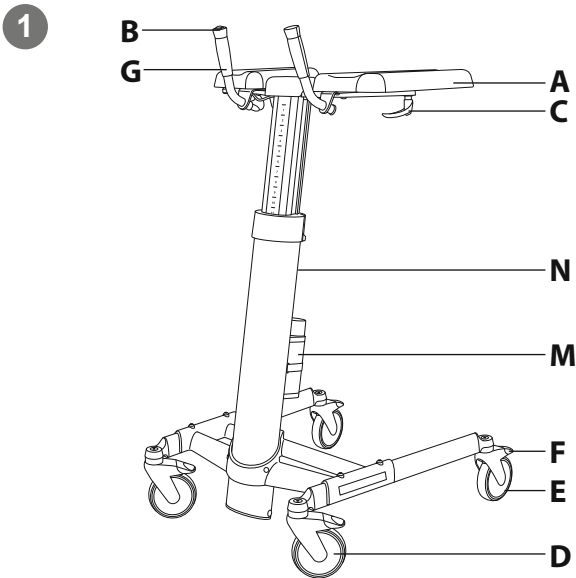
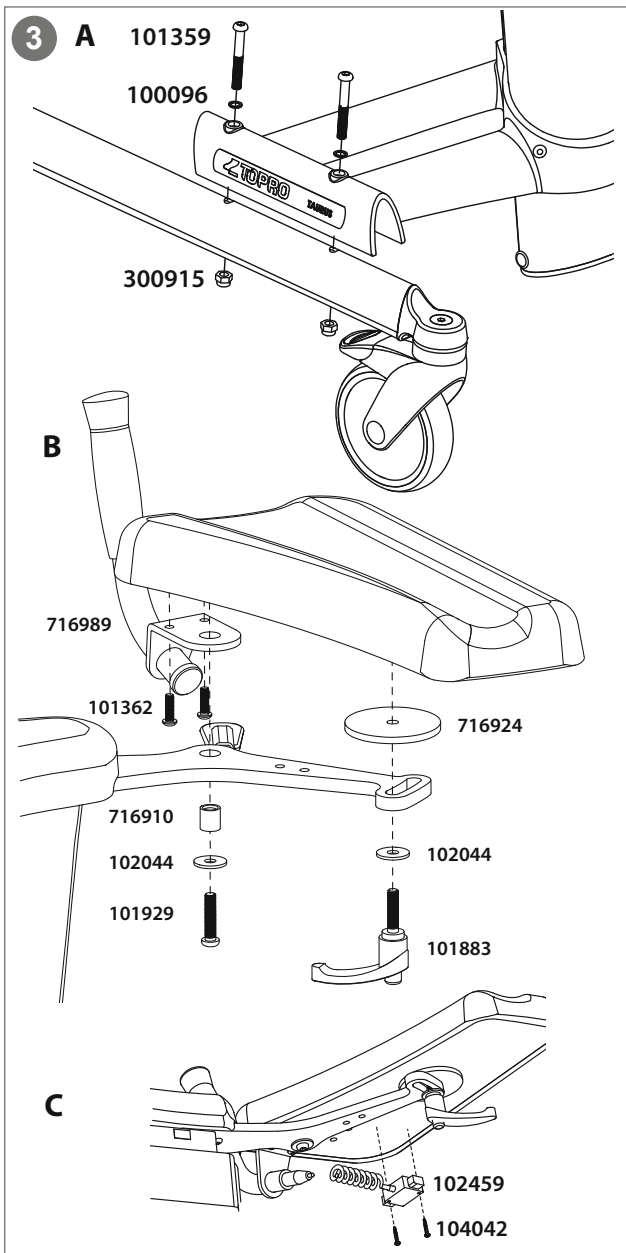
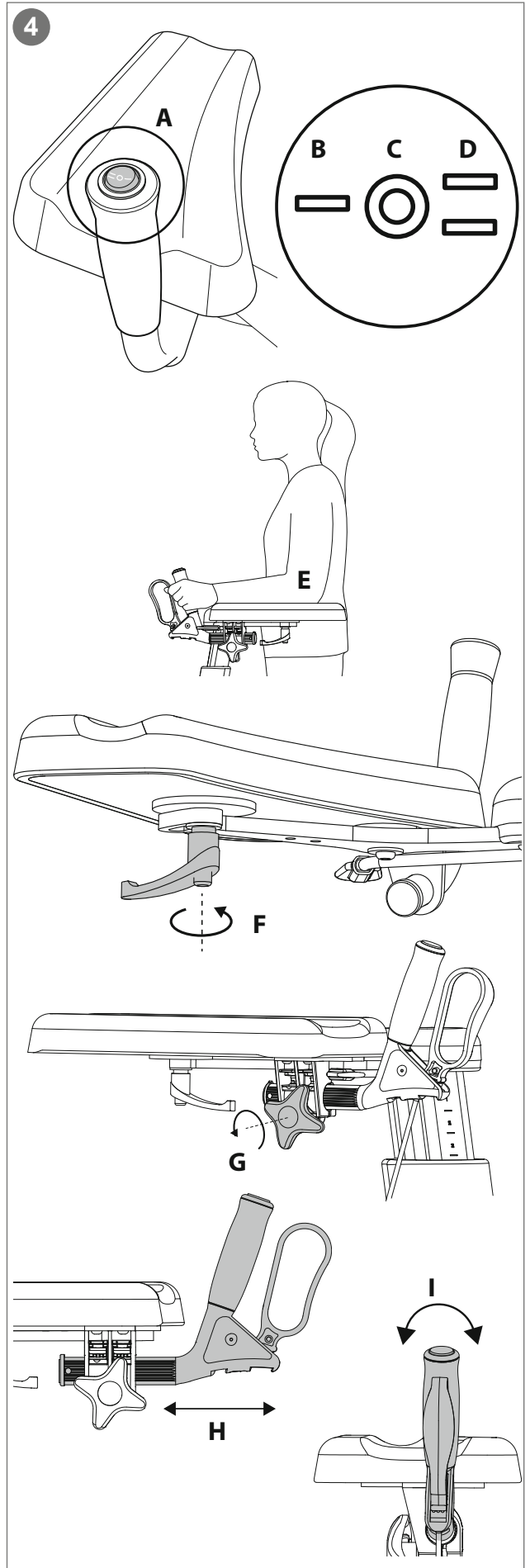
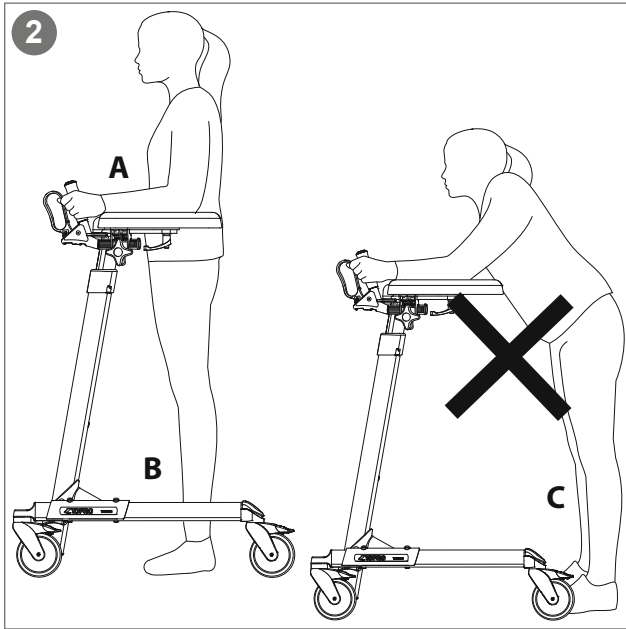
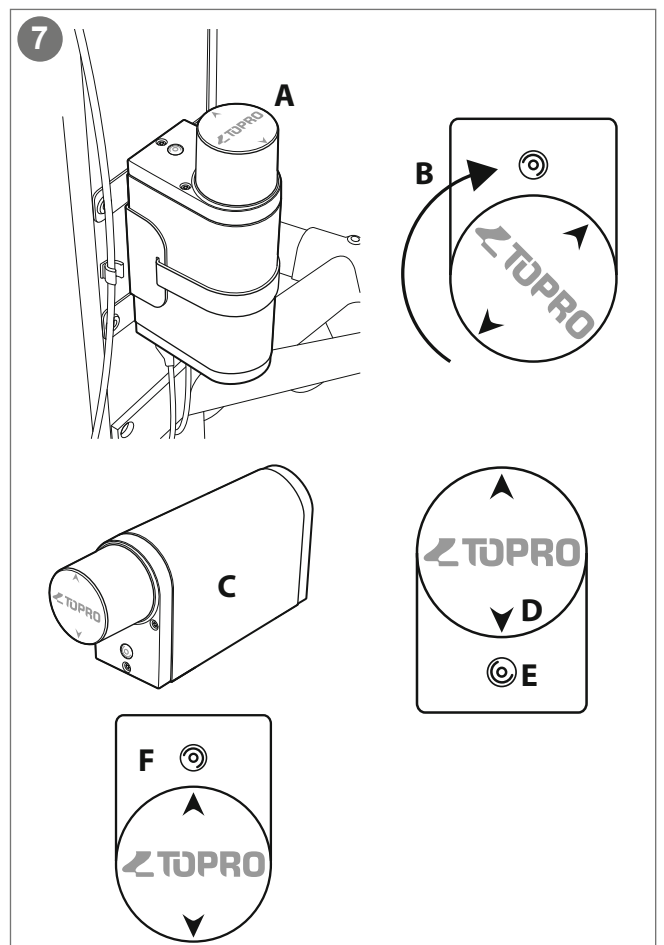
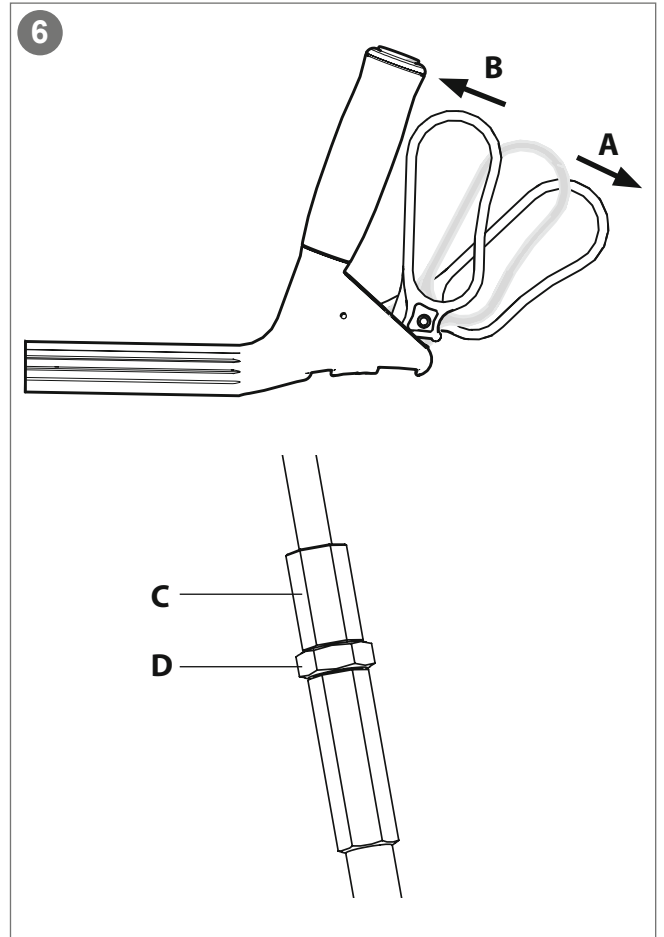
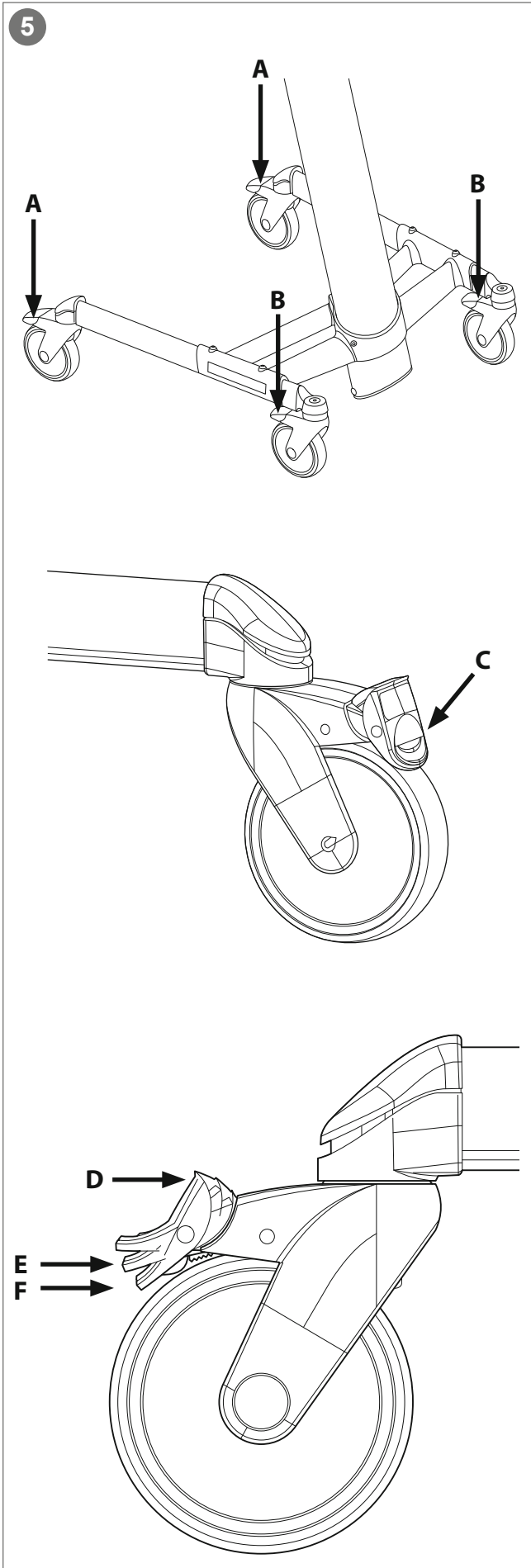
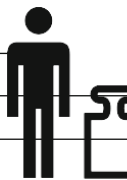
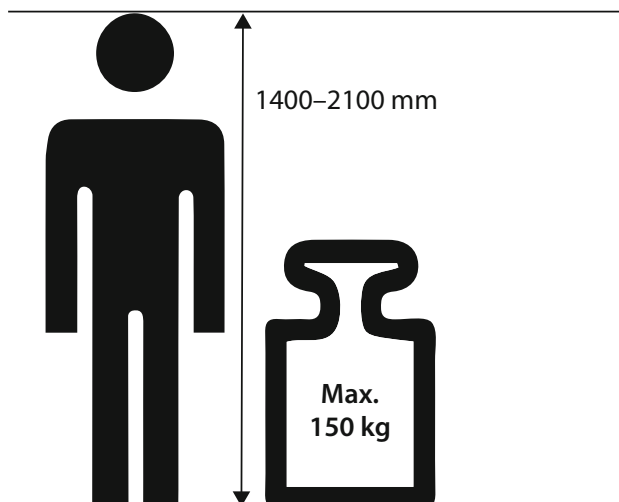
 — 22       — 23

**Congratulations with your choice of a new TOPRO forearm walker**

Every TOPRO forearm walker is developed, designed and manufactured in Norway. The product conforms to Medical Device Regulation (EU) 2017/745. It is tested and approved according to EN ISO 11199-3:2005, IEC 60601-1:2005 + A1:2012, IEC 60601-1-6:2010 + A1:2013, IEC 62366: 2007 + A1:2014, 60601-1-11:2015, IEC 60601-1-2:2014. On receipt of the goods, please check your product. If you have any queries, please contact your mobility shop or dealer immediately. If you find it challenging to read this user manual, an electronic version is available on [topromobility.com](http://topromobility.com).

**Intended use**

The TOPRO Taurus E is a forearm walker, which shall give support to users with reduced balance and/or reduced walking ability. The model TOPRO Taurus E Premium might also serve as raising support. The forearm walker is designed to be pushed, not pulled. The intended user is an adult. Height and weight restrictions apply. The main user of the product is the patient him-/herself; however, the walker can also be operated by an assistant. *Contraindications: The product is not suitable for persons with low strength in arms and legs, with very poor balance, or with considerable cognitive disabilities. The product is designed and approved for indoor use on plain foundation. We advise caution when using the TOPRO Taurus in private homes with narrow doorways, as the maximum width of the walker is 660 mm.*

To get to know the forearm walker and learn how to use it properly, training by a professional is recommended.

**Important**

This user manual contains important safety instructions and information regarding the use of the product and its accessories. In this manual, the user/patient is the person being supported. The assistant is the person who helps to control the raising/lowering of the forearm walker.

 **Caution!** This symbol indicates important information related to safety. Follow these instructions carefully.

 **Consult user manual before use!** It is important to fully understand the content of the user manual before attempting to use the equipment.

**0 The grey numbered circles in this user manual refers to the numbered information graphic and illustrations inside the cover**

**1 Get to know your TOPRO product**

- |  |  |
|--|--|
| <b>A</b> Forearm support                         | <b>H</b> Adjustable handles (Premium)        |
| <b>B</b> Height adjustment button                | <b>I</b> Hand brake (Premium)                |
| <b>C</b> Lever for adjustment of forearm support | <b>J</b> Brake cable (Premium)               |
| <b>D</b> Front wheels                            | <b>K</b> Lock wheel for the handle (Premium) |
| <b>E</b> Rear wheels                             | <b>L</b> Directional block (Premium)         |
| <b>F</b> Foot brakes                             | <b>M</b> Battery                             |
| <b>G</b> Handles (Basic)                         | <b>N</b> Product label                       |

**2 Safety** 

To give the right support and safety, walk straight and make sure the walker is kept as close to your body as possible (A). Walk between the rear wheels (B), not behind them (C).

Do not push the walker forward if the front wheels have stopped at an obstacle. The walker may tilt and cause an accident.

Do not use the walker on stairs. Be careful when passing thresholds etc.

Be aware that some parts of the walker might feel hot or cold when exposed to extreme temperatures.

Be aware: TOPRO Taurus E must not be used in a damp environment.

Be aware not to put your fingers in open holes or between stationary parts.

Check the brakes before use every time.

Do not modify the product as this might put your safety at risk and the guarantee will become void.

When stationary or parked, make sure the parking brakes on both sides are activated.

Do not use the walker to transport heavy loads or persons.

Do not use the walker on slippery surfaces.

Keep the walker away from fire and hot items.

Do not climb on the walker or let children play with it.

**3 Assembling and adjusting**

**TOPRO Taurus E is delivered with two levels of equipment:**

**Basic (814790)** Adjustable arm pads and brakes on all wheels. Delivered partly assembled. Assembling: Please follow the steps A to C in illustration 3.

**Premium (814789)** Has in addition to the basics adjustable handles and driving and parking brake on the handles. The rear wheels have a swing lock (directional block). Delivered completely assembled.

Before use, adjust the forearm supports and handles (see section 4), and charge the battery (see section 7).

#### 4 Height adjustment / Forearm supports / Handles

**Height adjusting:** Press the button (A) on top of the handle to the left or right to adjust the height. The marking on the button refers to its three positions where (B) = Down, (C) = No action, (D) = Up.

**Recommended height:** Stand with relaxed shoulders and position your elbows at a 90-degree angle (E). The level of the forearm supports should align to the level of your elbows. To help ensure a correct posture, a professional should help you with the adjustment of the forearm supports and the handles.

##### Adjusting the forearm supports:

The forearm supports can be adjusted individually. Unscrew the lever under the forearm support (F) until the cushion is moveable. Adjust to the required position and tighten the screw. *NB! If the lever protrudes, it can be dangerous. Pull it down and rotate it away to be safely hidden.*

##### Adjusting the handles (TOPRO Taurus E Premium):

Unscrew the lock wheel (G). Pull the handle (H) backwards or push it forwards and tilt it to the desired angle (I). After adjusting the handle, tighten the lock wheel (G). Be careful to not hurt your fingers in the process.

#### 5 Foot brakes / Directional block

##### TOPRO Taurus E Basic – all wheels / TOPRO Taurus E Premium – front wheels:

Step on the foot brakes on the rear wheels (A) and when required also on the front wheels (B), until you hear a clicking sound. The wheels are now in the locked position (C). To release them, step on the top of the foot brake pedal.

##### TOPRO Taurus E Premium – rear wheels:

Step on the foot brakes on the rear wheels (A) once until you hear a clicking sound. The wheels are now in the directional blocked position (E). Push the foot brakes further down until you hear a second click. The wheels are now locked (F). To release them, step on the top of the foot brake pedal (D).

#### 6 Hand brakes (Taurus E Premium)

Ensure the brakes are working properly before every trip. *Do not drive or push the forearm walker while the parking brakes are activated or when the wheels are partly locked using the brakes. This indicates incorrect use and can cause wear and tear, which makes the tires and brake quality weaker.*

**Parking brakes:** Push both brake levers forwards (A) to activate the parking brakes. Be aware that the lever has some resistance before it stops and locks the rear wheel. Pull both brake levers towards you to release the parking brakes.

**Driving brakes:** Pull both brake levers towards you (B) to reduce speed. It is a two-handed braking system, which means that the brake is activated for the rear wheel on the same side of the walker as the brake lever. When applying only one of the brakes, the walker will turn to the same side when walking. Releasing the lever will stop the braking. *Do not continuously use the brakes when the walker is in motion.*

**Adjusting / checking:** Loosen the nut (C). Tighten / loosen the brake cable with the adjustment screw (D). Ensure that the brakes are working properly and only touching the wheels when in use. Adjust the brakes evenly on both sides.

**Maintenance:** Inspect the brakes regularly by following the information above. For best possible grip, wipe off the tires with a damp cloth. The wheels and brakes can be replaced if they are worn out or damaged.

#### 7 Charging the battery

It is important to charge the battery regularly to extend battery life. We recommend charging the battery every 24 hours. Regular charging will improve the performance of the battery. It takes approximately 5–8 hours to fully charge the battery. Do not charge batteries at too high or too low temperatures.

**Charging:** Connect the charger to the power grid without the battery. There are two LED lights on the charger. The yellow power-on-LED lights up when the charger is connected to power. If there is no battery in the socket, the other LED light will not light up. Remove the battery (A) from the control box of the walker, by holding the battery and turning it 45 degrees to lift it out (B). Please check that the contact surfaces are clean. Place the battery in the charger (C) by sliding it in and rotating it

until it is pulled into place by the magnets. Check that one of the arrows on the battery (D) points towards the white and blue mark/eye (E). The LED light will now turn blue. Once the charging is complete, the blue LED will turn green, and it will remain permanently lit. To remove the battery from the charger, rotate it 90 degrees. It will then be ejected and be ready for use. To mount the battery on the walker again put the battery back in the control box on the column. Check that one arrow points towards the white and blue mark/eye and the other arrow points straight back (F).

**Tips and recommendations:** If the battery's safety thermostat is activated due to overload, the battery must be allowed to cool for at least half an hour before being charged or used again. A new battery may be inserted into the controller after one minute. Always wait at least two seconds after the actuator has stopped before removing the battery from the controller. Always use batteries which are in good condition and fully charged when inserted into the controller.

You can extend the lifetime of the battery by occasionally recharging it even though it is not fully discharged. If the battery level drops below the critical level and the system shuts down, you must wait at least one minute before inserting a different battery and using the system again. If the walker is not to be used for an extended period (several days or weeks), the battery should be removed from the controller to avoid deep discharge, which can cause lasting damage to it. Avoid getting metal shavings or other magnetic objects on the battery poles or in the charger.

#### Charge cycle and LED indications

LED	MODE
LED 1: YELLOW	Charger connected to power
LED 2: NO LIGHT	No battery in charger
LED 2: BLUE	Battery in charger, charging
LED 2: GREEN	Charging complete
LED 2: BLUE LIGHT FLASHING	Faulty battery, not charging. Remove battery.

**Be aware**

If a fault occurs, the blue LED will flash. This is then an indication that the battery is faulty and needs to be removed. The C3 charger will then not commit charging. When the battery level is very low, it might only be possible to adjust the height of the walker down, not up. The battery must be reset if this occurs. Take the battery out of the control box and press the red height adjustment button on the handle (see section 4; *height adjusting*) until you hear a beep.

Different beep signals:

Short “beep...beep” from the control box every 24th second = low battery level. Charge the battery.

Long “beeeeeep” when the battery is put in the control box = good connection.

Short “beep...beep” when the walker is being raised and lowered = low voltage. The battery should be replaced.

If recharged, the battery will still give power but only for a limited time.

Long “beeeeeep” when the Taurus is being raised and lowered = battery voltage lower than 17.6 V. It will only be possible to lower it, not raise it.

Short “beep...beep...beep...beep” means too high temperature inside the control box. Stop using until the control box has cooled down.

## Troubleshooting

### Brakes

Worn out / defect wheel	->	Replace the complete wheel
Not braking effectively	->	Adjust the brake cables
Worn out / defect brake mechanism on wheel	->	Replace the entire back wheel section
Worn out / defect brake handle	->	Replace the complete brake handle
Worn out / damaged brake cable	->	Replace the brake cable

### Height adjustment

Battery is flat/does not work	->	Charge or replace the battery (see section 7)
The height adjustment does not work	->	See section 7, charging the battery

### Height adjustment goes down, not up

The battery is not correctly mounted	->	Check that the battery is clean and correctly mounted
Loose cable contact	->	Check that all contacts are properly in place and tightly fitted
Damage on the cable	->	Check all cables and replace any damaged one(s)
Defect switch	->	Replace the switch
Defect height adjusting panel (accessory)	->	Replace the height adjusting panel

**If all the above has been checked and the product is still not operating correctly, then the cause may be damage to the control box or actuator. Please contact your supplier.**

## Transport

The walker can be transported either in vertical or in horizontal position. It should be set in the lowest height position to save space and facilitate handling. Ensure that the foot brakes are activated. Be careful when loading the walker into a vehicle, giving attention to securing it.

## Storage

The walker should be stored indoors and in an upright position. Do not place heavy items on top of it while stowed away. The maximum storage temperature is 50 °C. The walker must cool down for one hour at 25 °C before use. The minimum storage temperature is -20 °C. The walker must warm up three hours at 25 °C before use.

## Cleaning

The walker can be cleaned with a damp cloth and mild household detergents. Fluids should be avoided. Do not use abrasives, steam- or high-pressure cleaners. The walker cannot be hosed because of the electric motor. In case of multiple patients using the walker, it is necessary to clean and disinfect the walker and its accessories before use by another patient. Remove the battery from the walker before the disinfection/cleaning process begins. Allow the walker to dry in room temperature before use. Do not store below 0 °C before the walker is completely dry. Cleaning must be done manually without mechanical devices (e.g., ultrasonic cleaner or disinfectant washer). No further use of products in the case of water penetration.

## Disinfection

Disinfect by wiping generally accessible surfaces with disinfectant. Disinfection must only be carried out by authorized personnel and by wearing sufficient protective equipment. The surface must be cleaned with a disinfectant that contains 70–80 % ethanol. We advise against disinfectant that contains chlorine or phenol. Running liquids should be avoided. The battery of the walker must be removed before the disinfection process take place. Before use, allow the walker to air-dry in room temperature until there is no more superficial moisture on the product. *The manufacturer cannot be held responsible for any damage or injuries that might occur using harmful disinfectant or disinfection carried out by unauthorized personnel.*

## Inspection / Maintenance

### / Re-use

Do not service and maintain this medical electrical equipment while in use! The wheel bearings are sealed and maintenance-free. The parts do not need to be lubricated. It is recommended that inspection and maintenance are carried out regularly (frequency is determined by how and how often the walker is used). Check the following items: Tubes, frame, screws, cables, handles, forearm supports, brakes, wheels and accessories. This also applies when the walker is to be prepared for re-use. Please read detailed instructions about maintenance of brakes and cleaning/disinfection of the walker in this user manual.

## Material / Recycling

The walker is made of plastic-coated aluminium tubes, plastic couplings, push handles of polyamide-6 and TPE, and forearm cushions made of polyurethane. Most parts can be recycled. Dispose the walker and its packaging according to the applicable regulations in your country. The battery contains NiMH (nickel). Do not dispose the battery in ordinary household waste, some communities offer recycling or collection of batteries. For information, please contact your public authorities.

**Allergic Reactions:** No known allergic reactions.

**IP degree and definition:** Solid particle protection; >12.5 mm (fingers or similar objects.)

Liquid ingress protection: Vertically dripping water shall have no harmful effect when the enclosure is tilted at an angle of 15° from its normal position. For technical details on the electrical components, see the last section of the user manual.

## Warranty

TOPRO Taurus E is guaranteed free of faults and defects for 2 years. Damage caused by incorrect use or parts that are exposed to natural wear and tear (e.g., battery, brakes, wheels and handles) are exempt from the 2-year warranty. For repairs during the warranty period, please contact your local mobility shop or dealer. The warranty is void if unauthorised spare parts or accessories have been or are being used on the product. The expected lifespan of the product is estimated to be 10 years if it is used in the correct manner according to this user manual and its safety and maintenance instructions.

## 8 Product label

- 1 - Gross weight
- 2 - Safe working load
- 3 - Voltage
- 4 - Duty cycle
- 5 - Maximum user weight
- 6 - Maximum height of walker
- 7 - Maximum width of walker
- 8 - Manufacturer
- 9 - Model name
- 10 - Model number
- 11 - Date of manufacture
- 12 - Serial number
- 13 - Global Trade Item Number
- 14 - Caution! – Read Instruction for use
- 15 - Consult Instructions for use (blue icon)

- 16 - Intended for indoor use
- 17 - Medical Device
- 18 - CE marking (This product conforms to Medical Device Regulation (EU) 2017/745)
- 19 - Do not dispose in general waste
- 20 - Degree of IP protection
- 21 - GS1 DataMatrix

### Other symbols used on the product:

- 22 - Non-ionizing electromagnetic radiation. Equipment includes RF-transmitters. Interference may occur in the vicinity of equipment marked with this symbol.
- 23 - Class II ME Equipment, double insulated

## Accessories

	TOPRO Item No.
One-handed brake	814026
Forearm side support, pair	814647
Tray	814810
Small basket	814821
Footplate	814645
Infusion stand	814793
Stand for body fluid boxes	814034
Oxygen bottle holder	814820
Height adjustment panel	814646
Wall mount for the battery charger	814085

Accessories mounted on the walker may influence stability; we therefore advise you to use them with care. Accessories can be ordered separately to enable the forearm walker to be customised to individual needs. Contact your mobility shop, dealer or TOPRO for an updated overview of available accessories or visit our homepage [topromobility.com](http://topromobility.com).

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## Technical details electrical components

Controller box: C3con1-x, manufacturer Concens AS, Denmark  
 Actuator: Con 5, manufacturer Concens AS, Denmark  
 Battery: C3 NiMH 24 V – 1400 mAh  
 Battery charger: C3-CHG-01-XX-02, Input voltage: 100–240VAC, Class II, Input frequency: 50–60Hz, Maximum power rating: 50VA, Output voltage: 24VDC, Output current: Max 400mA, IP22.  
 Expected service life of the battery is 5 years under heavy use condition according to the manufacturer.

Duty Cycle: 10% maximum activation (on) time 2 minutes and the minimum deactivation (off) time 18 minutes.  
 Working ambient temperature for actuator: –5°C to +40°C  
 Working ambient temperature for controller and battery charger: +5°C to +40°C  
 Humidity: 20–70 %  
 Delivery and storing ambient temperature for actuator, controller and battery charger: –20°C to +50°C  
 Humidity: 20–70 %

### Guidance and manufacturer's declaration

#### Electromagnetic EMISSIONS for all ME EQUIPMENT and ME SYSTEMS

The TOPRO Taurus walking aid is intended for use in the electromagnetic environment specified below.

The customer or the user of the TOPRO Taurus walking aid should assure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment – guidance
RF emissions CISPR 11	Group 1	The TOPRO Taurus walking aid uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RF emissions CISPR 11	Class B	The TOPRO Taurus walking aid is suitable for use in all establishments, including domestic establishments and those directly connected to the public low voltage power supply network that supplies buildings used for domestic purposes.
Harmonic emissions IEC 61000-3-2	Complies	
Voltage fluctuations/ flicker emissions IEC 61000-3-3	Complies	

### Guidance and manufacturer's declaration

#### Electromagnetic IMMUNITY for all ME EQUIPMENT and ME SYSTEMS

The TOPRO Taurus is intended for use in the electromagnetic environment specified below.

The customer or the user of the TOPRO Taurus should assure that it is used in such an environment.

Immunity test	IEC 60601 test level	Compliance level	Electromagnetic environment – guidance
Electrostatic discharge (ESD) IEC 61000-4-2	± 6 kV contact ± 8 kV air	± 6 kV contact ± 8 kV air	Floors should be wood, concrete or ceramic tile. If floors are covered with synthetic material, the relative humidity should be at least 30 %.
Electrical fast transient/burst IEC 61000-4-4	± 2 kV for power supply lines ± 1 kV for input/output lines	± 2 kV for power supply lines ± 1 kV for input/output lines	Main power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	± 1 kV line(s) to line(s) ± 2 kV line(s) to earth	Main power quality should be that of a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	<5 % UT (>95 % dip in UT) for 0,5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 s	<5 % UT (>95 % dip in UT) for 0,5 cycle 40 % UT (60 % dip in UT) for 5 cycles 70 % UT (30 % dip in UT) for 25 cycles <5 % UT (>95 % dip in UT) for 5 s	Main power quality should be that of a typical commercial or hospital environment. If the user of the TOPRO Taurus requires continued operation during power mains interruptions, it is recommended that the TOPRO Taurus be powered from an uninterruptible power supply or a battery.
Power frequency (50/60 Hz) magnetic field IEC 61000-4-8	3 A/m	3 A/m	Power frequency magnetic fields should be at levels characteristic of a typical location in a typical commercial or hospital environment.

NOTE: UT is the a.c. mains voltage prior to application of the test level.

**Guidance and manufacturer’s declaration**

**Electromagnetic IMMUNITY for ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING**

The TOPRO Taurus is intended for use in the electromagnetic environment specified below.  
The customer or the user of the TOPRO Taurus should assure that it is used in such an environment.

IMMUNITY test	IEC 60601 TEST LEVEL	Compliance level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150 kHz to 80 MHz	3 V	Portable and mobile RF communications equipment should be used no closer to any part of the TOPRO Taurus, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.  Recommended separation distance $d=1,16\sqrt{P}$ $d=1,16\sqrt{P}$ 80 MHz – 800 MHz $d=2,33\sqrt{P}$ 800 MHz – 2,5 GHz

Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2,5 GHz	3 V/m	
------------------------------	----------------------------	-------	--

where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in metres (m). Field strengths from fixed RF transmitters, as determined by an electromagnetic site survey a), should be less than the compliance level in each frequency range b).

Interference may occur in the vicinity of equipment marked with the following symbol:

NOTE 1 At 80 MHz and 800 MHz, the higher frequency range applies.  
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

- a) Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and TV broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. If the measured field strength in the location in which the TOPRO Taurus is used exceeds the applicable RF compliance level above, the TOPRO Taurus should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the TOPRO Taurus.
- b) Over the frequency range 150 kHz to 80 MHz, field strengths should be less than 3 V/m.

**Recommended separation distances between portable and mobile RF communications equipment and the ME EQUIPMENT or ME SYSTEM for ME EQUIPMENT and ME SYSTEMS that are not LIFE-SUPPORTING**

The TOPRO Taurus is intended for use in an electromagnetic environment in which radiated RF disturbances are controlled. The customer or the user of the TOPRO Taurus can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF communications equipment (transmitters) and the TOPRO Taurus as recommended below, according to the maximum output power of the communications equipment.

Rated maximum output power of transmitter W	Separation distance according to frequency of transmitter m		
	150 kHz–80 MHz $d=1,16\sqrt{P}$	80 MHz–800 MHz $d=1,16\sqrt{P}$	800 MHz–2,5 GHz $d=2,33\sqrt{P}$
0,01	0,12	0,12	0,23
0,1	0,37	0,37	0,74
1	1,16	1,16	2,33
10	3,67	3,67	7,37
100	11,6	11,6	23,3

For transmitters rated at a maximum output power not listed above, the recommended separation distance d in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (W) according to the transmitter manufacturer.

NOTE 1 At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.  
NOTE 2 These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.

**TOPRO products are developed, designed and manufactured in Norway by:**

## **TOPRO Industri AS**

Rambekkveien 5, NO-2816 Gjøvik, Norway  
(+47) 61 13 46 00  
info@topromobility.com  
www.topromobility.com

### **Subsidiaries**

#### **TOPRO GmbH**

Bahnhofstraße 26 d  
82256 Fürstentfeldbruck  
GERMANY  
Tel: +49 (0) 8141 88 89 39-0  
Fax: +49 (0) 8141 88 89 39-22  
cs@topro.de  
www.topromobility.com

#### **TOPRO GmbH**

Zweigniederlassung Stetten  
Im Stetterfeld 2  
5608 Stetten  
SWITZERLAND  
Tel: +41 (0) 44 940 36 50  
Fax: +41 (0) 44 940 36 51  
mail@topro.ch  
www.topromobility.com

#### **TOPRO Mobility B.V.**

C'park Bata  
Europaplein 1  
Gebouw 21 – unit 2.17  
5684 ZC Best  
THE NETHERLANDS  
Tel: +31 (0) 6 27 288 801  
cs@topromobility.nl  
www.topromobility.com

#### **TOPRO Limited**

Trent Business centre  
Thoroton Rd  
West Bridgford  
Nottingham  
NG2 5FT  
UNITED KINGDOM  
Tel: +44 (0) 115 846 5406  
Fax: +44 (0) 844 739 16 15  
info@topro.co.uk  
www.topromobility.com

#### **TOPRO Mobility AB**

Buskuddsvägen 9  
185 95 Vaxholm  
SWEDEN  
Tel: +46 (0) 8 311 330  
info@topromobility.se  
www.topromobility.com

#### **TOPRO Mobility DK**

Jadevej 11  
8541 Skødstrup  
DENMARK  
Tel +45 (0) 53 763 131  
info@topromobility.dk

Distributor / dealer

Learn more

topromobility.com